

## User guide for TRP-WR-JS

### Before use

- Please remove all packaging and ensure the product has not been damaged in transit.
- Carefully read the instructions and warning texts before using the product
- Keep instructions for reference
- On each product you will find a label that shows the maximum permitted load.

### Contents

- Fixings - 12 x nuts, bolts, washer's and spring washer's (fig1), Joining support plate x1 (fig 2), Joining support legs x4 (fig 3), support brackets x2 (fig 4).

**Fig.1**



**Fig. 2**



**Fig.3**



**Fig.4**



## To Assemble

1. Attached the support legs to the joining support plate using the fixings provided, set legs to the required height. Leave the fixings loose and place the ramp in to position to allow for any fine adjustments. (figs. 5,6)
2. Once set in the correct position, tighten all fixings on the support legs securely with a spanner (spanner not provided). (fig. 7)
3. Attached the support brackets to the top of support legs on both sides using the fixings provided and tighten all fixings securely with a spanner (spanner not provided) (fig.8,9)
4. The joining support is now assembled. If required, the joining support can be fixed to the ground (fixings not provided). Position the joining support accordingly and the ramps can now be placed onto the support. Please note ramp to ground MUST rest on narrow section and ramp to top or rise height MUST rest on deeper section (fig. 10)

Fig. 5



Fig. 6



**Fig. 7**




**Fig. 8**



**Fig. 9**



Fig. 10

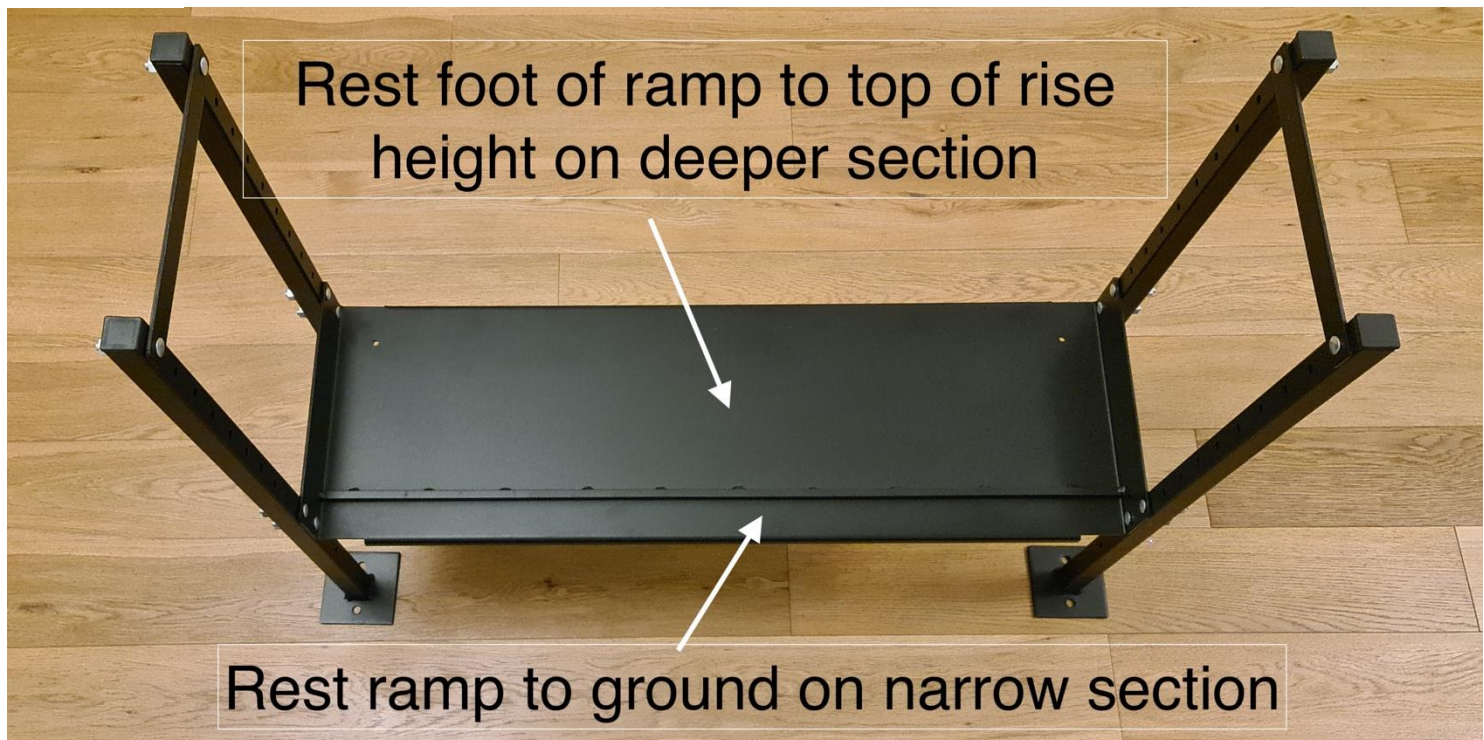

### Maintenance

- If the product becomes damaged in any way it should not be used (contact supplier/manufacturer).
- The product may be cleaned with normal cleaning products and left to dry before use.

### Warranty

The Ramp People warrants that the ramp/accessory is free from defects in material and workmanship.

This warranty shall remain in effect for 1 year from the date of original consumer purchase. Use other than the published/intended item will void the warranty.

The warranty does not cover failure of the unit during the warranty period if the failure is not caused by defect in material or workmanship or if the failure is caused by unreasonable use.

The Ramp People are not liable for incidental or consequential injury or damage to property.

Should it be necessary to return the ramp, it is essential it is adequately packed to protect it in transit, preferably in its original box; otherwise The Ramp People do not accept any responsibility for transit damage to the unit.

In some cases, the items will be supplied by 3<sup>rd</sup> party seller, please keep the original invoice. This does not affect your statutory rights.

**The TRP-WR-JS** is part of our range of products from The Ramp People. To view the full range please visit our website

**[www.theramppeople.co.uk](http://www.theramppeople.co.uk)**

Or contact us via, post, email and/or telephone.

## **The Ramp People**

Unit C3  
Sandown Ind. Park Mill Road  
Esher  
Surrey  
KT10 8BL

Tel: +(44) 1372 478960

Email: [hello@theramppeople.co.uk](mailto:hello@theramppeople.co.uk)

[www.theramppeople.co.uk](http://www.theramppeople.co.uk)