

# User Instructions

## Economy folding Ramp

### TRP607-Range



Product Code SKU	Length	Usable Width	Overall Width	Product weight	Maximum load height (1:6)	Capacity
TRP607M-2	610mm	715mm	720mm	5.4kg	100mm	300kg
TRP607M-3	914mm	715mm	720mm	8kg	150mm	300kg
TRP607M-4	1219mm	715mm	720mm	10.4kg	200mm	300kg
TRP607M-5	1500mm	715mm	720mm	13.2kg	250mm	300kg
TRP607M-6	1829mm	715mm	720mm	15.8kg	300mm	300kg
TRP607-7	2130mm	715mm	720mm	19kg	355mm	275kg
TRP607-8	2440mm	715mm	720mm	23kg	455mm	275kg

*\*Product weights are approximate and may vary.*

#### **General Safety Warning**

**Please ensure these instructions are fully read and implemented. Failure to do so may result in injury to the user. Retain in a safe place for future reference.**

#### **Intended Use**

This folding ramp, TRP607- is designed to provide access for manual or powered wheelchair and mobility scooters (and their carers, if appropriate) over thresholds, steps and curbs. There are various available length versions in the range, see table above for size information

#### **Unpacking**

- No tools are required to assemble this product. (However if the ramp is to be secured to the landing surface, then appropriate tools will be required)
- Remove item from packaging; Check for damage. If you see any, or suspect a fault, do not assemble or use the product; contact your supplier immediately.

#### **Safety Precautions**

When choosing a ramp, the length, and therefore the angle of the ramp depend upon:

- The height to be raised
- The type of wheelchair or scooter
- The weight of the wheelchair or scooter and the user
- The strength of the helper or user.

As a general guide, a minimum gradient of 1 in 6 is suggested, although 1 in 8 is recommended, when the user is seated on the wheelchair or scooter.

For loading an unmanned wheelchair or scooter into a vehicle, a higher gradient can be used, but this should not exceed 1 in 3.3 In addition to this, it is important that you refer to your vehicle's user guide for the proper degree of incline/decline and direction before using the ramp.

#### **- Never exceed its recommendations.**

- Only carry the ramp using the carry handles. Take care when lifting; follow standard manual handling procedures.
- Take particular care when using the ramp in wet/damp conditions.

## Instructions for installation and use

1. Remove all parts from the packaging and check the ramps are free from damage.



2. Carry the ramp to the required area using the carry handle and place one end of the ramp on top of the step/ Threshold/ Platform, so the top of the ramp is higher.



3. Place the ramp on the landing surface ensuring that the top lip is resting securely on the landing surface. Fold out the ramp so it is as shown in the picture.



4. The top of the ramp has a lip to allow secure and comfortable operation of the ramp. Ensure this lip is fully on the to surface (as shown in the image below).



5. Once the ramp is unfolded, Ensure the top lip is secure before use, once checked the ramp will be ready to use.



6. After use, unfold the ramp and remove from top surface. When locked, the ramps can be carried safely. (transport bags not included but available to purchase.).

7. Do not exceed the maximum load or Height.

## Important Note:

The surface holding the top lip needs to be substantial; make sure that the surface and supporting structure is of sound construction.

To prevent the ramp from becoming dislodged during use, ensure the top lip overlaps the landing surface as far as possible.

## Care and Maintenance

- Inspect ramp regularly for signs of wear or damage.
- Do not use if the ramp is damaged.
- Clean regularly, wipe over with a solution of mild detergent and water. Remove any grit that may have become caught between the sections.
- Please note the folding wheelchair ramp is not designed to be left outside, please store inside
- Only for use as a mobility aid, not suitable for loading goods, trolleys and similar

**The Economy Wheelchair Ramp Range** are part of our range of ramps from The Ramp People.

To view the full range please visit our website

**[www.theramppeople.co.uk](http://www.theramppeople.co.uk)**

Or contact us via, post, email and/or telephone.

### **The Ramp People**

Unit 5  
Quantum Business Park  
Beacon Hill Road,  
Church Crookham  
Fleet  
GU52 8EA

Tel: +(44) 1372 478960

Email: [hello@theramppeople.co.uk](mailto:hello@theramppeople.co.uk)

[www.theramppeople.co.uk](http://www.theramppeople.co.uk)

## Warranty

The Ramp People warrants that the ramp is free from defects in material and workmanship.

This warranty shall remain in effect for 1 year from the date of original consumer purchase.

The warranty does not cover failure of the unit during the warranty period if the failure is not caused by defect in material or workmanship or if the failure is caused by unreasonable use.

The Ramp People are not liable for incidental or consequential injury or damage to property.

Should it be necessary to return the ramp, it is essential it is adequately packed to protect it in transit, preferably in its original box; otherwise The Ramp People will not accept any responsibility for transit damage to the unit.

In some cases, the items will be supplied by 3<sup>rd</sup> party seller, please keep the original invoice.

This does not affect your statutory rights.