


# Setup Instructions

Read before using your Replacement Feet


- 1 Using a Phillips screwdriver, remove screw (by turning counterclockwise) from center of each of three (3) HurryCane® feet. Then remove rubber feet and discard the old feet and screws.

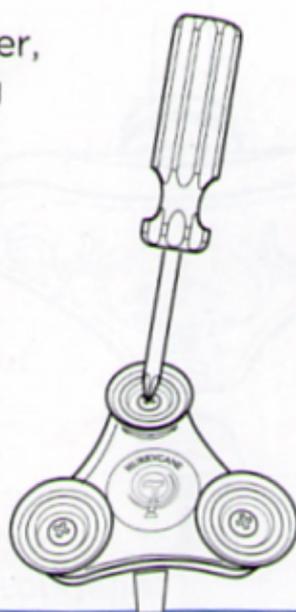

- 2 Carefully place new rubber foot onto pivoting head foot cavity.

Slide metal washer onto screw. Place screw with washer through rubber foot and into the threaded hole of pivoting head foot cavity. Be sure to tighten screw securely.

Repeat this step for the remaining two feet. HurryCane® requires the feet to be replaced every six months.

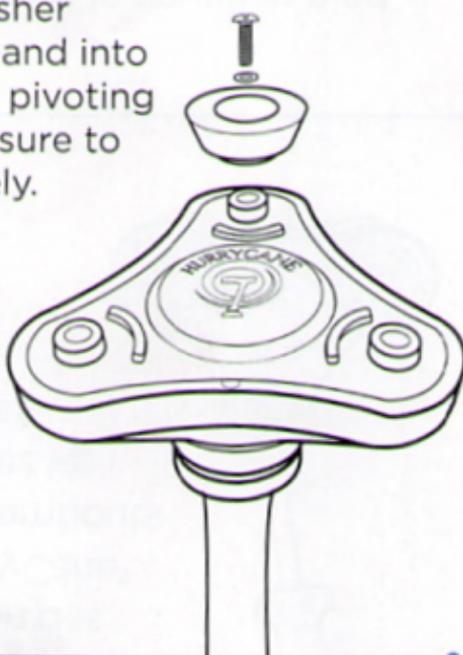

## Comments, Feedback, or Questions

Toll-Free Number: 1-800-933-1136

E-mail: [succes@hurrycane.com](mailto:succes@hurrycane.com)

Like us on Facebook: [facebook.com/AllTerrainCane](https://www.facebook.com/AllTerrainCane)



©2019 HurryWorks LLC  
99 Seaview Boulevard  
Port Washington, NY 11050  
[www.hurrycane.com](http://www.hurrycane.com)  
Made in China

REV2.8.15.19

HC-FEET



8 22383 57107 2